

# For Safe & Easy Measurements of Urine Specific Gravity!!

**ATAGO®**

DIGITAL HAND-HELD "PEN"  
URINE S.G. REFRACTOMETER

**Pen-Urine S.G.**

Cat.No. 3741

The PEN-Urine S.G. utilizes the refractive index method to measure the specific gravity of urine. The specific gravity of urine varies according to concentration.



Actual size

## Features

- Calibration with water only
- Measurement in 2 seconds
- Extremely water resistant (IP65)
- Light & Compact, 70g
- Ergonomically designed for easy, one-handed operation
- External Light Interference (ELI)
- Convenient storage case
- Automatic Temperature Compensation (ATC)

## Specifications

Measurement range	: Urine S.G. 1.0000~1.0600
Resolution	: Urine S.G. 0.0001
Measurement accuracy	: Urine S.G. ±0.0010
Measurement temperature	: 10~40°C (ATC)
Measurement time	: 2 seconds
Power supply	: AAA alkaline battery × 1
Battery life	: Approx. 8,000 measurement (When an alkaline battery is used)
International Protection Class	: IP65 Water resistant (Prism head:IP67)
Dimensions and weight	: 160(W) × 38(D) × 18(H)mm, 70g

## The PEN type refractometer is extremely easy to use!

Fast measurement results: safe and hygienic.

**Dip!**



OR



**Touch!\***

### Measurement Methods

Dip the tip into the sample and press the START key OR press the START key (and within 10 seconds) touch the tip into the sample. The measurement value will be displayed in 2 seconds.

\*Small Sample Volumes

**Quick & Easy Cleanup!**



All ATAGO refractometers are designed and manufactured in Japan.



**HACCP GMP GLP**

ATAGO products comply with HACCP, GMP, and GLP systems standards.

### ATAGO CO.,LTD.

Headquarters: 32-10, Honcho, Itabashi-ku, Tokyo 173-0001 Japan  
TEL: 81-3-3964-6156 FAX: 81-3-3964-6137  
overseas@atago.net <http://www.atago.net/>

### ATAGO U.S.A., Inc.

12011 NE First Street, Bldg. C, Suite 110, Bellevue, WA 98005 U.S.A.  
TEL: 1-425-637-2107 FAX: 1-425-637-2110  
customerservice@atago-usa.com <http://www.atago.net/>

### ATAGO INDIA Instruments Pvt. Ltd.

A-501, Mangal Aarambh Business Centre, Near Kora Kendra, Off. S.V. Road, Borivali (West), Mumbai-400 092 India  
TEL: 91-22-2833-8038 / 8076 FAX: 91-22-2899-8164  
customerservice@atago-india.com



※Specifications and appearance are subject to change without notice.

**NOVA-TECH**  
INTERNATIONAL

800 Rockmead Dr Ste 102 • Houston, TX 77339-2496  
Tel: (281) 359-8538 • Toll Free Tel: (866) 433-6682  
Fax: (281) 359-0084 • Toll Free Fax: (866) 433-6684  
sales@novatech-usa.com • www.novatech-usa.com